

INSTRUCTIONS FOR USE



















ENG

SOFT CONTACT LENSES

A guide to trouble-free soft contact lens wear

Monthly replacement tinted and coloured soft contact lenses.
MATERIAL: Methafilcon A 45%, water 55% in sterile buffered saline.

SYMBOL REFERENCE GUIDE

	Use by date: Refer to packaging		Manufacturers details
	Batch code: Refer to packaging		Made in Australia
	Consult instructions for use		Indicates the CE conformity marking and the notified body number
	Sterilised using steam or dry heat		Prescription only product
	Do not use if packaging is damaged		Authorised representative in the European community
	Do not resterilise		Power
	Providing UV protection		Diameter
	Recyclable		Base curve
	Use caution		
	Temperature range limit		

Always wash and dry your hands before handling your lenses. Do not use if the packaging or product is damaged. Please read all instructions prior to use.

LENS INSERTION

For each eye, ensure the lens is not inside out (See Fig. 1 & Fig. 2). Place on your finger to verify the shape. Using your other hand and second finger, hold up the upper lid and hold down the lower lid. Place the lens on the eye (See Fig. 3). Repeat the process for the other eye.

LENS REMOVAL

Look up and hold down the lower lid. Gently pinch the bottom edge of the lens and lift it out of your eye with the pads of your index finger and thumb (See Fig. 4). Repeat the process for the other eye.

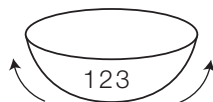



Fig. 1 

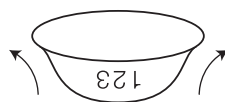
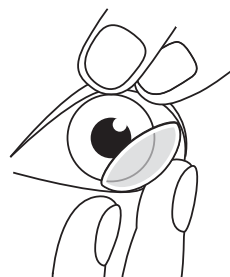

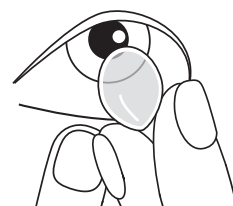



Fig. 2 



LENS INSERTION
Fig. 3 



LENS REMOVAL
Fig. 4 

DISINFECTION/SOAKING

Clean and disinfect your lenses strictly following the instructions on your solution bottle or as directed by your eye-care practitioner.

If the lenses dry out follow the disinfection/soaking step above, replace with new lenses or consult your eye-care practitioner.

PRECAUTIONS

Any variation to the following should only be performed under the advice of your eye-care practitioner.

- It is recommended that you replace your lenses frequently and care for them daily. Your eye-care practitioner will recommend a wearing regime. Follow their recommendations and make regular visits.
- These lenses are designed for daily wear and must be replaced as directed.
- Take care when inserting the lens as a scratched eye is vulnerable to infection.
- Discard damaged or scratched lenses immediately.
- Insert lenses before applying make-up. Remove lenses before removing make-up. Use fibre and oil free mascaras. Avoid contact between your lenses and any cosmetic products including aerosols.
- Keep lenses out of reach of children.
- Remove lenses in the presence of noxious or irritating fumes.
- Consult your eye-care practitioner before using your lenses for any water related activity.

WARNINGS

- DO NOT sleep while wearing your lenses.
- DO NOT use household products (e.g. disinfectants) on your lenses.
- DO NOT wear your lenses when using eye medications or eye drops (unless advised to do so by your eye-care practitioner).
- DO NOT use saliva to wet your lenses.
- DO NOT rinse your lenses or lens case with water.
- DO NOT expose your lenses to water while you are wearing them (e.g. showering.)

ADVERSE REACTIONS

If you notice any of the following issues with your contact lenses then **IMMEDIATELY REMOVE YOUR LENSES**.

- Eye stinging, inflammation, burning, itching or eye pain
- Decreasing comfort from when you first inserted the lens
- Excessive watering of the eye
- Unusual eye secretions or redness
- Rainbows or halos around objects
- Sensitivity to light
- Dry eyes

If damage to the lens is identified upon inspection, do not put the lens back in your eye. You should discard the lens and insert a new lens into the eye.

If the lenses have a foreign body on them and appear undamaged, follow the disinfection/soaking step above, then re-insert them. If the problem continues, remove the lenses and immediately consult your eye-care practitioner.

ALL ENQUIRIES



Manufactured by Gelflex Laboratories.



52 Mulgool Road, Malaga

Western Australia, 6090

www.gelflex.com

EC REP ISEMED srl

Via P. Togliatti 19/X
40026 IMOLA (BO) - Italy

C € 2460

CONTROL NUMBER F1094-03
Correct as of April, 2020.

Gelflex 
Australian contact lenses