




Gelflex 
Australian contact lenses

PROFESSIONAL FITTING GUIDE

POST GRAFT 1 & 2 LENSES

 (03) 9792 3127 · 1800 335 559

 (03) 9793 1635

 orders@gelflex.com

 www.gelflex.com

APPG1 & APPG2 CONTACT LENSES

GELFLEX - POST GRAFT 1 & 2

The APPG1 and APPG2 Post Graft lens set is a lens designed for the post graft patient.

SPECIFICATIONS

- Base curve 7.00mm to 8.40mm
- Overall diameter 11.20mm
- Back optic diameter 9.00mm
- Tricurve design
- APPG2 - mid peripheral curve 0.05mm flatter than APPG1.

LENS FITTING RECOMMENDATION

As post grafted corneas are often quite distorted, it is not possible to obtain a perfect fit. A balanced fitting lens is recommended.

